CS Connect

Installationsvägledning

Meddelande

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The screenshot displays the CS Connect web application interface. At the top left, the 'CS Connect' logo is visible. A search bar contains the text 'Search a case number, a patient or partner'. The main header shows the case ID 'EFA-0019' and navigation icons for search, user profile, settings, and help. Below the header, a sidebar on the left lists 'Unsubmitted' cases, with 'EFA-0019 Unsubmitted' selected. The main content area shows the case details for 'EFA-0019', including a 3D dental model image. The status is 'Unsubmitted'. Metadata includes: Partner (To be defined), Submission date (Type: Model), Modality (CS 3500), Acquisition date (16/01/2014 19:57), and Gender (Date of birth). A bottom navigation bar contains 'Summary | Attached files | Notes | History'. The main content area below the navigation bar displays a message: 'No summary to display. Summary available once case is submitted.' At the bottom left, a status bar shows '1 cases' and 'Last update few seconds ago'.

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First connection ✕

Welcome > Practitioner > Practice > Terms of use

Welcome on CS Connect!
You must fill in your profile and accept service terms and conditions.

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First connection ✕

Welcome > Practitioner > Practice > Terms of use

You may edit you profile in application settings. Fields marked with * are required.

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
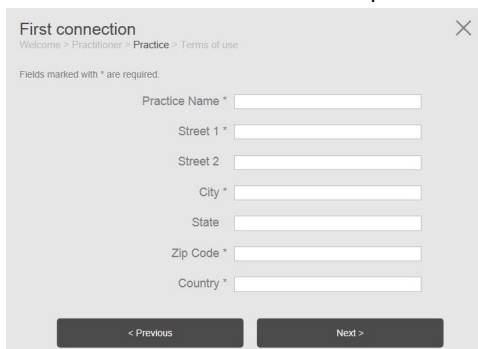
The screenshot shows a web form titled "First connection" with a breadcrumb trail: "Welcome > Practitioner > Practice > Terms of use". Below the title, it says "Fields marked with * are required." The form contains several input fields: "Practice Name *", "Street 1 *", "Street 2", "City *", "State", "Zip Code *", and "Country *". At the bottom, there are two buttons: "< Previous" and "Next >".

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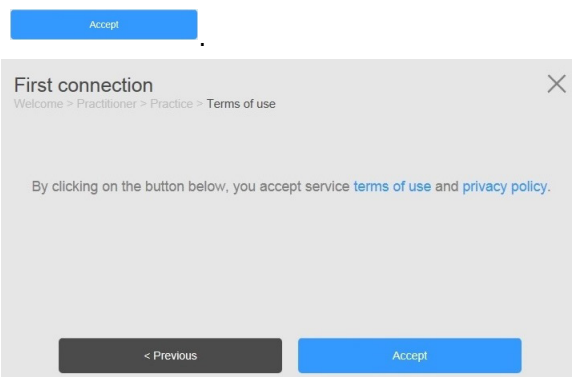The screenshot shows the "Terms of use" screen. It has the same breadcrumb trail as the previous screen. The main text reads: "By clicking on the button below, you accept service [terms of use](#) and [privacy policy](#)." Below this text, there are two buttons: "< Previous" and a blue "Accept" button.

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The screenshot shows the CS Connect web application interface. At the top, there is a search bar with the text "Search a case number, a patient or p". Below the search bar, there are two main panels. The left panel says "You have no case." and the right panel says "No case selected." with a grey square icon. At the bottom, there is a status bar that says "0 cases" and "Last update few seconds ago".

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